


MYO/Lead Disposable Patient Cable

with locking safety slide bar
to prevent unintentional
electrode release



Catalog Number
019-530

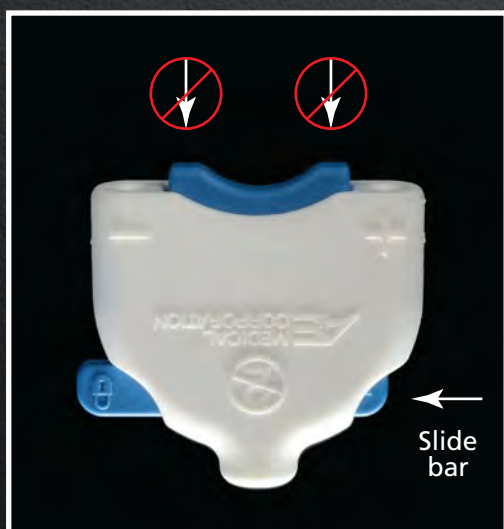

The electrode release button cannot be
depressed with safety locking slide bar
in locked position as shown.



Catalog Number
019-535




MYO/Lead Disposable Patient Cable

with locking safety slide bar to prevent unintentional electrode release

To Insert Electrodes:

- Unlock slide bar if needed. The electrode release button cannot be pressed in locked position.
- Push down and hold "B" nearest the desired well "A."
- Insert electrode completely into the well. Release "B."
- Repeat b) and c) for second electrode.

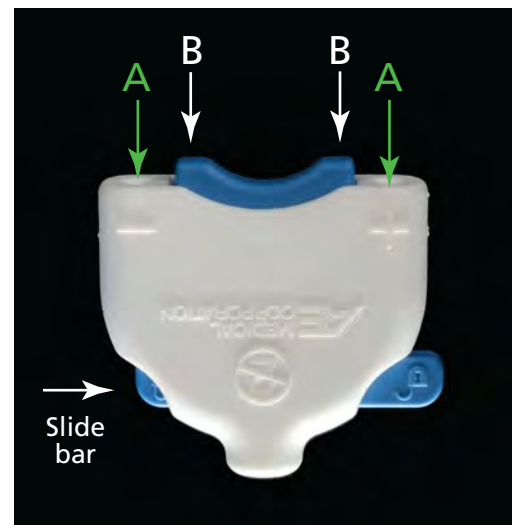

Locking Connector and Connecting:

- Lock slide bar.
- Connect plug to external equipment.



To Remove Electrodes:

- Unlock slide bar.
- Press and hold "B" nearest well.
- Remove electrode.

Ordering Information

<u>Catalog Number</u>	<u>Color</u>	<u>Description</u>
019-530	White	MYO/Lead Disposable Patient Cable Length: 1.8 meters (6 feet) Eight (8) cables per box.
019-535	Blue	MYO/Lead Disposable Patient Cable Length: 1.8 meters (6 feet) Eight (8) cables per box.



5206 ASBURY RD., P.O. BOX 758 • FARMINGDALE, NJ 07727 • USA

(732) 938-2266 • (800) 323-4035 • (732) 938-2399 FAX